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EN - Vitcone Orthopedic srl was born in order to develop, produce and sell the angular stability Vitcone® osteosynthesis orthopedic system. An innovative system that allows an easy removal of implants, the reduction of surgical times in fractures, and the simple and modulable infragmentary compression. The search for innovation has also affected the rational shape of orthopedic plates, pursuing the maximum biomechanical reliability with the right size. The skills and passion put

together are those of a veterinary orthopedic surgeon dr. Andrea Urizzi and the senior technicians of two precision mechanics companies in partnership with each other: Vitzani srl, Domegge (BL) and Ookii-Matrix, Rasai (BL), engaged for thirty years in the medical field (implants and instruments). The screw plate conical coupling, a cornerstone of the Vitcone® system, has been the subject of technological evolution since 2002. The essential instruments and the other devices useful for the orthopaedist in everyday work are also produced.

Both companies are UNI EN ISO 9001:2008 certified. Vitzani srl also has certification UNI EN ISO 13485:2012, aimed at human medical products, required for obtaining the CE marking according to the directive 93/42/CEE and S.M.I. The machines are an absolute technological pride. The materials to be implanted are certified. The processing tools are also certified and equipped with safety data sheets for monitoring the metal contamination of the product. The surface treatment and washing processes, which the two components of veterinary orthope-

dic implant undergo before being packaged are the forefront. The Vitzani company has a Cleaning room, validated and monitored according to the standard required by UNI EN ISO 14644-1, with UV-ray pax box both at the infeed and outfeed of the product, that regularly provides sterile implants for the human. For the veterinary market the consumable orthopedic material has the same level of surface finish and cleanliness but is not supplied sterile.

standard required by UNI EN ISO 14644-1, with UV-ray paxboxes both input as output of the product. These procedures are capable of breaking down the microbial load still present in the plants, ensuring a non-sterile but clean product.

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tions: the easy removal of the implants, the significant reduction in surgical times in reducing fractures and the possibility of inducing, just as pratically, a verifiable interfragmentary compression. The reliability, the mechanical rational in every detail. The hole in the plate, with the same screw diameter, is minimal; the screw can be inserted on plates of different thicknesses therefore, in the catalog, there are rigid plates and flexible plates. The shape of the plates and the distribution of the holes are designed according to the dictates of building science: maximum hold with minimum

The Vitcone® system is an orthopedic internal fixation system with angular stability, which involves the conical coupling between the truncated conical portion of the screw head and the complementary truncated

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EN - The system peculiarity is the upper part of the screw head that protrudes on the plate: it has been designed for the guick and safe removal of the screw and for the grip of the bone head during the reduction of the fractures. This dilated end has a lefthanded helix thread on the outside; like all screws, it houses the recess for the screwdriver at the top. In the event that the extraction of the screw with the screwdriver fails, this external thread in the head allows the Vitcone® extractor, simple device (Fig. 2) to easily remove the tenacious screw. The extractor has, inside the cannulated stem, a first section with a left-handed nut screw (Fig. 3). It screws onto the thread of the screw head, that is the protruding part (on the plate) of a coupled screw. The tip of the extractor thus comes into contact with the plate. Screwed further, the extractor causes an axial force on the screw towards the operator, which decuple sit from the plate itself. Continuing to rotate, the extractor acts like a screwdriver: it drags the screw out of the operating field (Fig. 4) allowing easy and safe removal of the screw, then of the orthopedic. The extractor, screwed onto the head of the screw, allows the

attachment of a second device belonging to the system: the Vitcone® plate tensioner (Fig.6) which is used to carry out interfragmentary compression or distraction of the fracture heads. The amount of compression between the bone heads, which is applied with the Vitcone® plate tensioner can be decided and can be decided and is measurable by applying a common torque meter to the knob of the same (Fig.7). In the fractures, torque values were applied in a range from 4 to 8 newton meters, depending on the size of the subject and the bone district. In this renard, a study on the optimal values to be applied is lacking, while it appears certain that the infragmentary compression, induced with the tensioner and maintained by applying angularly stable screw with the instruments in situ in compression, reduces the healing times of fractures and osteotomies.

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Figura 3 - Extractor tip 1 left nut screw turn; 2 threaded part of the screw head with left thread turns complementary to nut screw 1; 3 stroke of the extractor.

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FIGURA 6 THE VITCONE® PLATE TENSIONER serves to implement interfragmentary compression and distraction of the fractures heads.

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Figura 7 - In the knob of the Vitcone® plate tensioner there is the bolt head: the attachment for a common torque meter. It is therefore possible to decide and measure the newton meters applied during interfragmentary compression.

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Figura 4 - Extraction of a Vitcone® screw. The extractor is screwed onto the threaded edge of the screw head

hits the plate, unscrewing it further uncouples the

screw and take sit out of the operating field.

FIGURA 5 - with the extractor screwed on the head of the screw up to the stop on the plate, it fuses a handle: it allows the bone head to be moved towards the reduction.

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